<p>BRAEBON MEDICAL CORPORATION</p>	
<p>Unites States of America: 102-2981 Ford Street Extension Ogdensburg, NY 13669-3474 1.888.462.4841</p>	<p>Canada: 1-100 Schneider Road Kanata, ON, Canada, K2K 1Y2 01.613.831.6690</p>
<p>AUTHORIZED EUROPEAN REPRESENTATIVE</p>	
	<p>CEpartner4U BV Esdooinaan 13 3951 DB Mearn The Netherlands Tel: +31.343.442.524</p>
<p>© 2008 BRAEBON® Medical Corporation</p>	
<p>en</p>	
<p>Respiratory Effort Belt User Guide</p>	
<p>All Models</p>	
<p>Intended Use</p>	
<p>The BRAEBON MEDICAL CORPORATION Ultima Respiratory Effort Sensor™ is intended for use during sleep disorder studies as a qualitative measure of respiratory effort for recording onto a data acquisition system. The sensor uses piezo-electric technology and plugs directly into the electrode headbox.</p>	
<p>Warning: Respiratory effort sensors are for diagnostic purposes only and are not intended as apnea (apnoea) monitors.</p>	
<p>Caution: US Federal law restricts this device to sale by or on the order of a physician. You may wash the belt, but not the piezo-electric sensor</p>	
<p>Note: Refer to www.braebon.com for more information. Product specifications are subject to change without notice.</p>	
<p>How to Use</p>	
<p>Refer to your sleep system user guide for manufacturer’s recommended settings. Red 1.5 mm touchproof connector is positive; black is negative.</p>	
<p>General recording system settings:</p>	
<ul style="list-style-type: none">Low frequency filter: 0.1 Hz (TC=1 sec or longer) High frequency filter: 5 Hz to 15 Hz 50 / 60 Hz notch filter: optional Sensitivity / Gain: 2 mV/cm or 4,000 gain (increase or decrease as needed)	

Adjust the Ultima Respiratory Effort Sensor™ to fit the patient comfortably. Place the belt snugly around either the patient’s thorax or abdomen. The thoracic belt should sit just below or above the nipple line and the abdominal belt should sit below the rib cage.

To use the loop connector, place one end of the belt through the loop and fold to fasten the Velcro onto the belt. Ensure that the belt is neither excessively tight nor loose.

To use the buckle connector, simply fasten one end of the belt into the other end of the belt. Ensure that the belt is neither excessively tight nor loose.

Specifications

Operating Temperature: 5°C (41°F)-32°C (90°F)

Storage Temperature: 0°C (32°F)-60°C (140°F)

Operating/Storage Humidity: 10–95% Non-condensing

Disinfection and Cleaning

Caution: You may wash the belt, but not the piezo-electric sensor. Do not steam autoclave this device.

The Ultima Respiratory Effort Sensor™ is classified as a non-critical item because it only contacts clothing and intact skin. You may machine wash the respiratory belt, but not the piezo-electric sensor. If necessary, you may use isopropyl alcohol pads to gently clean the sensor. For additional information, refer to the APIC guidelines for selection and use of disinfectants (*American Journal of Infection Control*, Vol. 18, number 2, April, 1990, p. 99-117).

Ensure that The Ultima Respiratory Effort Sensor™ is completely dry before using. Never immerse the piezo-electric sensor enclosure at any time. Steam autoclaving is not recommended.

Warranty

BRAEBON MEDICAL CORPORATION warrants to the first consumer that the respiratory effort sensor (the “Sensor”), when shipped in its original container, will be free from defective workmanship, performance and materials and agrees that it will, at its option, either repair the defect or replace the defective Sensor or part thereof at no charge to the purchaser for parts or labor for a time period of one year from the date of purchase. The warranty described herein shall be the sole and exclusive warranty granted by BRAEBON MEDICAL CORPORATION and shall be the sole and exclusive remedy available to the purchaser. Use of the Sensor constitutes total and complete acceptance of this warranty. Correction of defects, in the manner and for the time period described herein, shall constitute complete fulfillment of all liabilities and responsibilities of BRAEBON MEDICAL CORPORATION to the purchaser with respect to the Sensor and shall constitute full satisfaction of all claims, whether based on contract, negligence, strict liability or otherwise. In no event shall BRAEBON MEDICAL CORPORATION be liable, or in any way responsible, for any loss of revenues or damage, direct, incidental, or consequential, including property damage, loss of profit, or personal injury resulting from the use or misuse of, or the inability to use this product. Nor shall BRAEBON MEDICAL CORPORATION be liable, or in any way responsible, for any damages or defects in the Sensor which were caused by abuse, misuse, tampering, neglect, incorrect battery type, or repairs or attempted repairs performed by anyone other than an authorized service provider.

<p>de</p>	
<p>Deutsch</p>	

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